

Voice Commands

Each time you want to give a voice command, you must activate the “Listening” prompt that appears inside the headset.

To activate the “Listening” prompt:

- Press and hold button 2 on the headset OR
- Press button 2 twice OR
- Press the trigger button on the back of the remote

When the “Listening” prompt appears, release the button and immediately speak the voice command. The “Listening” prompt will wait for a few seconds and then disappear if it doesn’t hear a command.

Below is a list of commonly used voice commands.

Changing the Mode

- “Welcome”
- “Scene”
- “Television”
- “Reading”
- “Inverted”
- “YouTube”
- “RP”

- “Settings”

Adjusting Magnification

- “Increase magnification”
 - “Zoom in”
- “Decrease magnification”
 - “Zoom out”
- “Magnification 5”
 - “Zoom 5”
 - Any number between 1-10

Adjusting the Picture

- “Increase Exposure”
 - “Decrease Exposure”
 - Adjust the exposure settings in Television View.
- “Increase Brightness”
 - “Decrease Brightness”
 - “Brightness 10”
 - Any number from 1-10
 - Default is 10
- “Increase Contrast”
 - “Decrease Contrast”
 - “Contrast 70”

- Any number from 1-100
 - Default is 50
- “Increase Field of View”
 - “Increase FOV”
 - “Decrease Field of View”
 - “Decrease FOV”
 - “Field of View 65”
 - Any number between 20-100
 - Default is 90
- “Right lens off”
 - “Right lens on”
 - “Left lens off”
 - “Left lens on”
 - Use in Welcome view
- “Turn On Flashlight”
 - “Turn Off Flashlight”

System Commands

- “Increase Volume”
 - “Decrease Volume”
 - “Volume 15”
 - Any number 1-15
 - “Max volume”
- “System Info”
 - Hear system information like software version, battery life, and others.

- “Increase IPD”
 - “Decrease IPD”
 - “IPD 58”
 - Any number from 50-70
 - Use in Welcome to correct double vision or feeling cross-eyed
- “Audio Feedback On”
 - “Audio Feedback Off”
 - Toggle audio announcements
- “Increase Audio Feedback Speed”
 - “Decrease Audio Feedback Speed”
 - Adjust the speed of audio announcements
- “Reset”
 - Reset any view back to default settings

Powering Off the Device

- “Shut down”
 - “Power off”
- “Sleep”